


Background

The enclosed NC404TS is pre-assembled with the monitor stand. The monitor stand will need to be removed and reassembled with the provided bracket and screws. Please follow the disassembly and re-assembly instructions below.

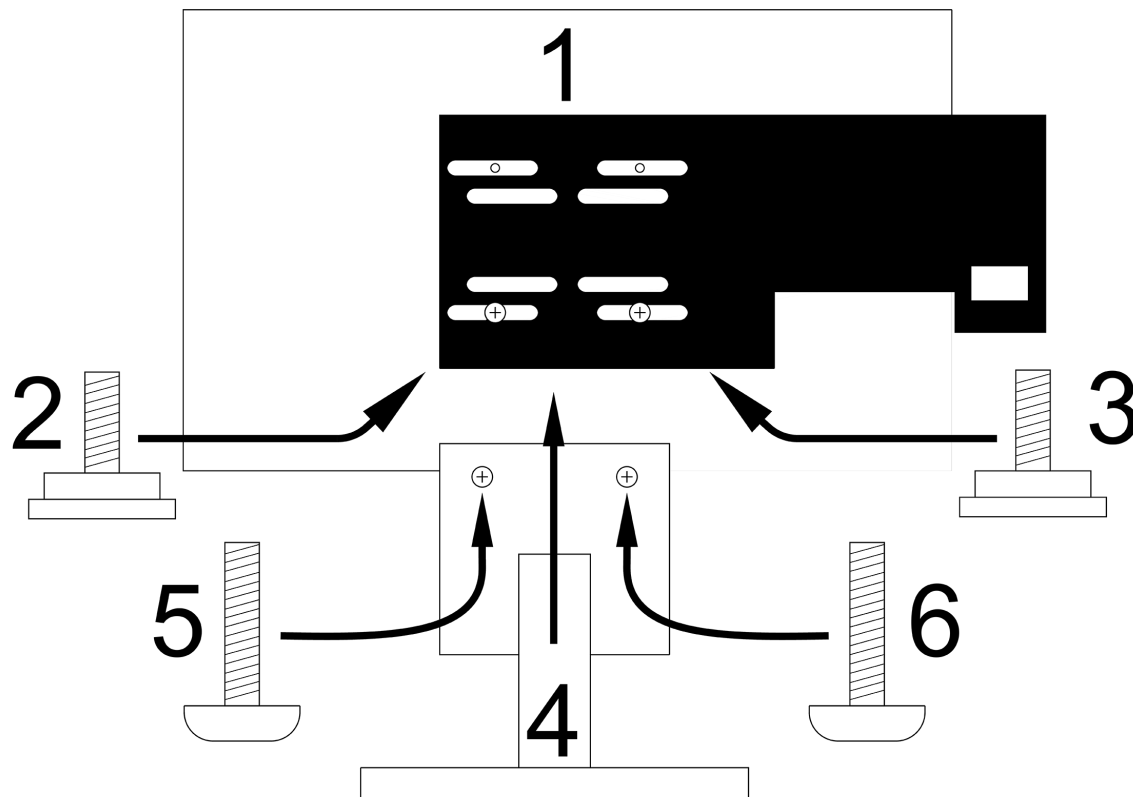

DISASSEMBLY

TOOLS NEEDED: PHILLIPS SCREWDRIVER

MONITOR WILL LIFT FREE AFTER REMOVING TWO SCREWS VISIBLE ON REAR

REMOVE MONITOR FROM BASE

REMOVE (AND RETAIN) TWO BLACK THREADED STANDOFFS (FRONT OF MOUNTING PLATE)

RE-ASSEMBLY (SEQUENCE SHOWN ABOVE)

PLACE MONITOR SCREEN-DOWN ON SMOOTH, FLAT SURFACE

ATTACH BRACKET (1) TO MONITOR AS SHOWN, USING BLACK THREADED STANDOFFS (2-3)

ATTACH BASE (4) TO MONITOR+BRACKET USING TWO #4-14 SCREWS PROVIDED BY TEKSTONE (5-6)